GEBRAUCHSANLEITUNG
USER'S MANUAL

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02-23

REF Artikel /
Article

26790

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DESCRIPTION

The Headrest

The Headrest can be retrofitted to the taillight mount (hole spacing 64 mm) of any HASE BIKES trike.

The height and angle of the Headrest is adjustable. It can be used for supporting the neck or the head (with or without side wings). See pages 6.

Intended use

Users are those who tend to let their head fall to the side or need a stable head position.

Contraindications

The headrest is unsuitable for all applications, which deviate from the intended use and which could lead to hazards. This use for example as a headrest in a car or operation by untrained persons. The headrest must be adjusted to suit the user.

Please observe the contraindications specified in the user manuals of the recumbent bikes and trikes.

Protection against environmental influences

The Headrest is made with corrosion-resistant materials. Anodized aluminum parts, plastics, and stainless steel are used.

Cleaning and disinfection

The Headrest can be cleaned and disinfected using common household cleaning and disinfecting agents.

Applicable standards and guidelines

This is an accessory within the meaning of Regulation 2017/745 on medical devices, which as such is CE-marked.

Service life

Depending on the frequency of use, a service life of up to 8 years can be expected if the product is used in accordance with the user manual.

Reuse

Under normal conditions of use, this accessory can be reused by a new owner after inspection by an authorized dealer.

Disposal instructions

At the end of its useful life, the product should be disposed of properly, in accordance with the local waste disposal regulations (i.e. not as household waste).

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SAFETY INFORMATION

The head restraint is only intended to support the head position of persons in the trike and must not be used in a car, for example.

Defective or damaged Harnesses may not be used and must be immediately replaced.

Information regarding incidents

Any serious incident* that occurs in connection with the product must be reported to the manufacturer and the responsible competent authorities of the member state in which the user and/or patient resides.

* A "serious incident" is any incident that had, could have had, or could have one of the following consequences, either directly or indirectly: a) the death of a patient, user, or other person, b) the serious temporary or permanent deterioration of the health condition of a patient, user, or other person, c) a serious risk to public health.

Residual risks

None known

Whom to contact with questions or problems

If you have any problems or questions about your therapeutic bike or trike, please contact your authorized dealer or HASE BIKES for assistance.

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


MOUNTING TO THE KETTWIESEL

1. Attach the Headrest mount **(01)** to the taillight mount, and the taillight **(02)** to the Headrest mount **(01)**, and secure the components using 4x M5 flat-head bolts **(03/04)**.

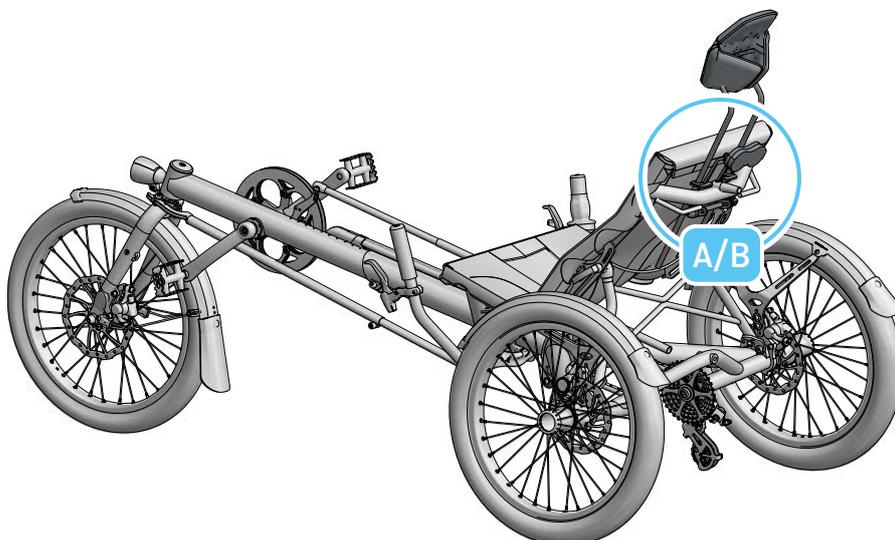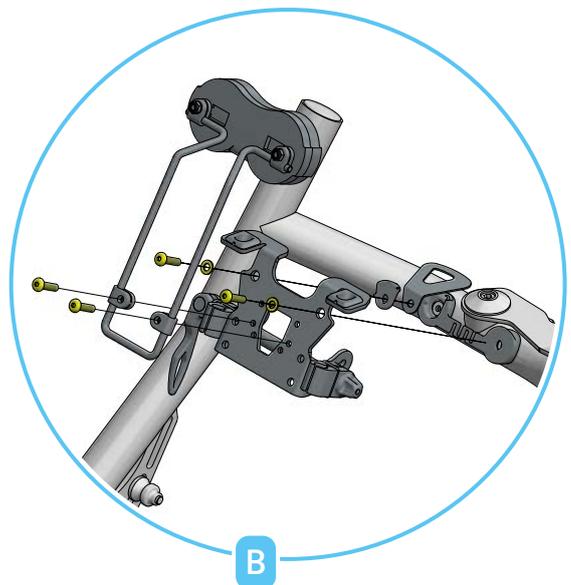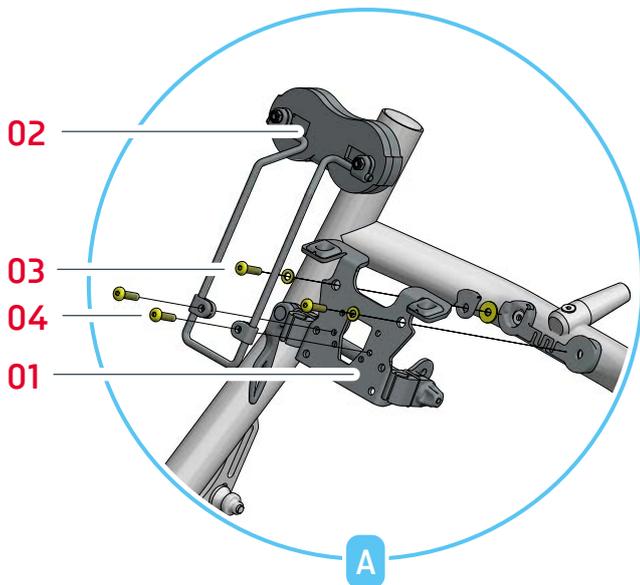(i) NOTE! Diagram **A** shows how the Headrest is mounted to a KETTWIESEL with no suspension system. Diagram **B** shows how the Headrest is mounted to a KETTWIESEL with a suspension system and a parking-support stand

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


MOUNTING TO THE TRIGO/TRETS

1. Attach the Headrest mount **(05)** to the taillight mount, and secure it using 2 M5 socket-head bolts **(06/07)**.
2. Mount the taillight **(08)** to the Headrest mount **(05)**, and secure it using 2 M5 socket-head bolts **(09)**.

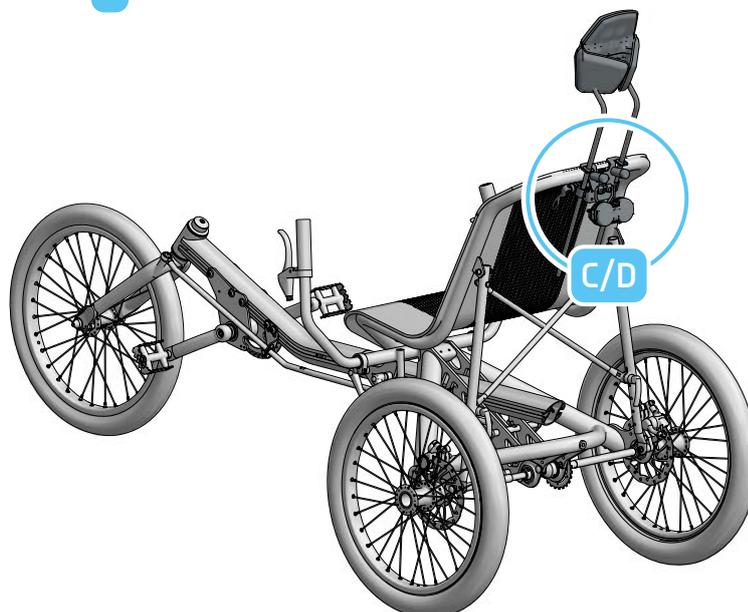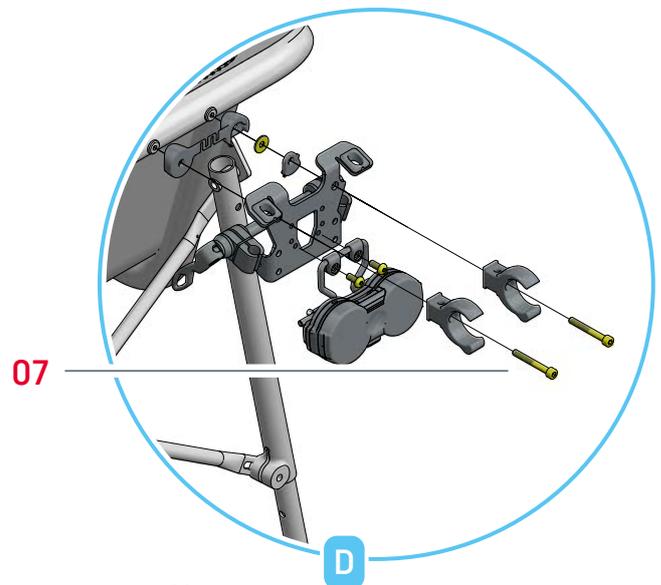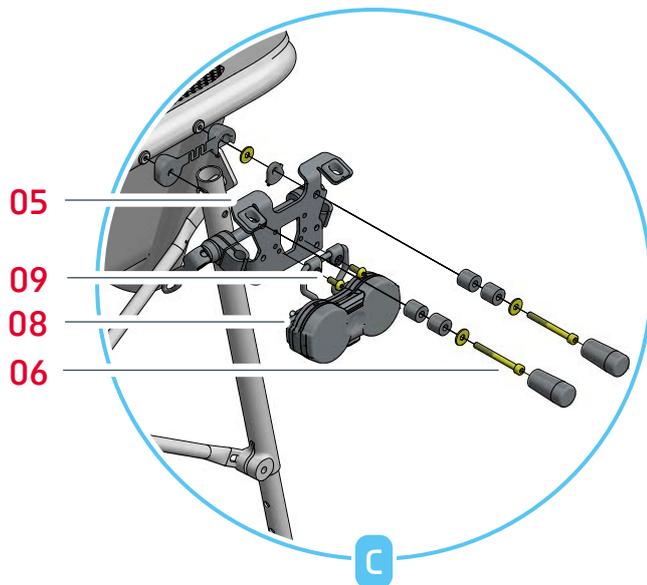i NOTE! Für M5 x 50 socket-head bolts **(06)** are used for mounting the Headrest to the TRIGO (see Diagram **C**). M5 x 45 socket-head bolts **(07)** are used for mounting the Headrest to the TRETS (see Diagram **D**).

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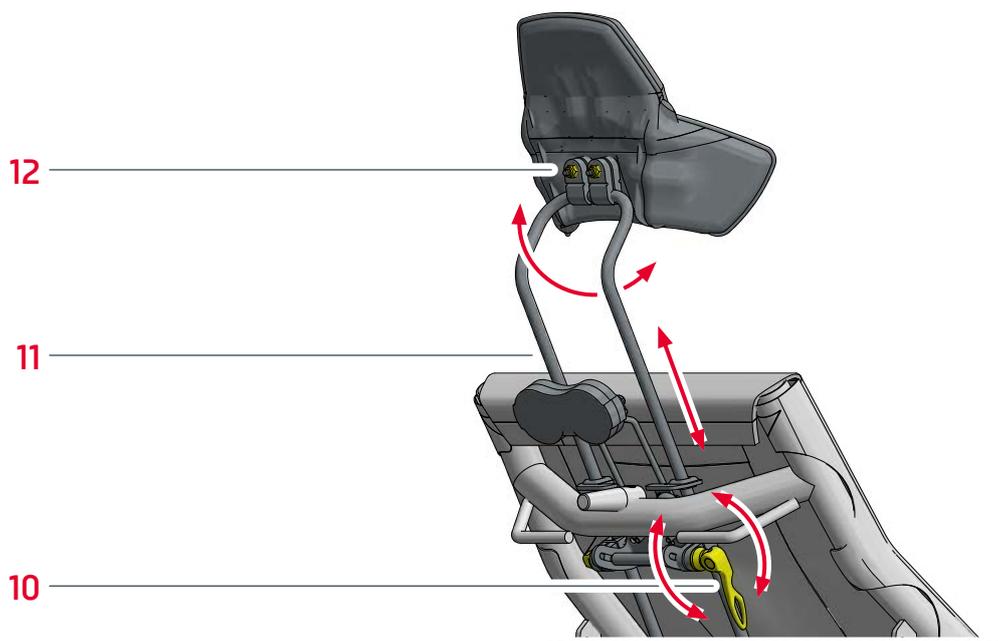
ADJUSTING THE HEADREST


1. Open the quick-release fastener **(10)**, and adjust the angle and height of the Headrest **(11)**.
2. Close the quick-release fastener **(10)**.
3. Loosen the 2 x M6 nuts **(12)**, and adjust the angle of the headrest.
4. Retighten the 2 x M6 nuts **(12)**.

i **NOTE!** The Headrest can be adjusted for supporting the neck or the head (with or without side wings).

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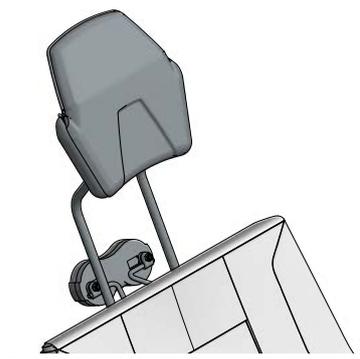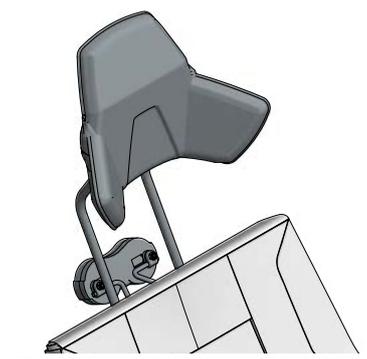
Adjustment for head sup-
port with side wings