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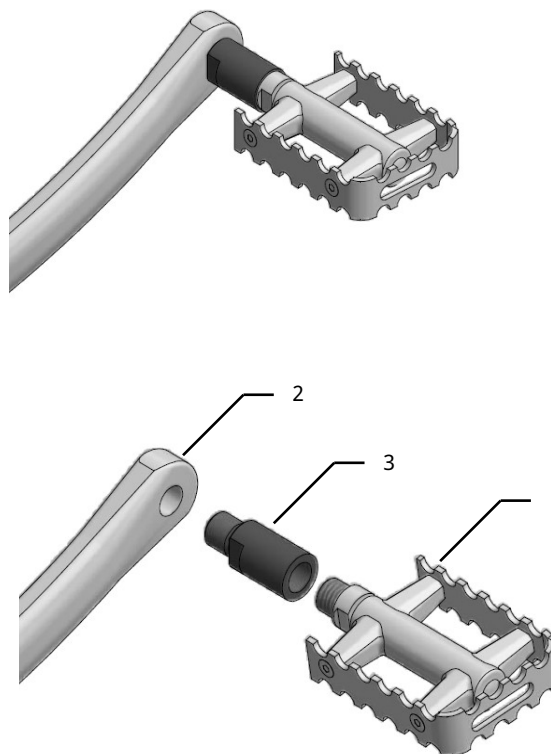
Pedal Axle Extension

Pedal Axle Extension

The Pedal Axle Extension can be mounted between the crank arm and pedal to reposition the pedal 30mm (1¼ inch) further away from the crank.

i Note: The right pedal has a right-hand thread, and the left has a left-hand thread. This must be taken into consideration for choosing the correct Pedal Axle Extension.

i Note: Pedals that can only be attached and removed using an Allen wrench are not compatible with Pedal Axle Extensions.

Installation

1. Remove the pedal (1) from the crank arm (2) with a wrench.
2. Screw the Pedal Axle Extension (3) into the crank arm (2).

i Note: Tighten the Pedal Axle Extension (3) with a torque of 35 Nm (26 ft-lbs).

3. Screw the pedal (1) into the hole of the Pedal Axle Extension (3).

i Note: Tighten the pedal with a torque of 35 Nm (26 ft-lbs).

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Pedal Axle Extension

The Pedal Axle Extension can be mounted between the crank arm and pedal to reposition the pedal 30mm (1 3/16") further away from the crank.

Intended use

The Pedal Axle Extension is intended for use by riders with very wide shoes or with externally or internally rotated feet or hips, who require an extension of the pedal axle to create sufficient distance between the pedals and crank arms.

Contraindications

For persons with very short legs, the Q Factor may be too wide if a Pedal Axle Extension is installed. The Pedal Axle Extension is not suitable for any purposes that are contrary to its intended use or that could be hazardous. Please observe the contraindications specified in the user manuals of the recumbent bikes and trikes.

Safety information

Defective or damaged Pedal Axle Extensions may not be used and must be immediately replaced.

Reuse

Under normal conditions of use, this accessory can be reused by a new owner after inspection by an authorized dealer.

Protection against environmental influences

The Pedal Axle Extension is made with corrosion-resistant materials.

Cleaning and disinfection

The Pedal Axle Extension can be cleaned and disinfected using common household cleaning and disinfecting agents.

Applicable standards and guidelines

This is an accessory within the meaning of Regulation 2017/745 on medical devices, which as such is CE-marked.

Information regarding incidents

Any serious incident* that occurs in connection with the product must be reported to the manufacturer and the responsible competent authorities of the member state in which the user and/or patient resides.

* A "serious incident" is any incident that had, could have had, or could have one of the following consequences, either directly or indirectly: a) the death of a patient, user, or other person, b) the serious temporary or permanent deterioration of the health condition of a patient, user, or other person, c) a serious risk to public health.

Whom to contact with questions or problems

If you have any problems or questions about your therapeutic bike or trike, please contact your authorized dealer or HASE BIKES for assistance.

Residual risks

None known

Service life

Depending on the frequency of use, a service life of up to 8 years can be expected if the product is used in accordance with the user manual.

Disposal instructions

At the end of its useful life, the product should be disposed of properly, in accordance with the local waste disposal regulations (i.e. not as household waste).