

Die Schiebebügel wurde für die Verwendung an allen Hasebikes Dreirädern (außer für Trets) entwickelt. Bei Verwendung an anderen Fahrzeugen übernehmen wir keine Haftung.

Montage

- 1. Lösen Sie den oberen Klettverschluss des Sitzbezuges. Entfernen Sie die Endstopfen aus den Rohrenden.
- Stecken Sie die Adapterhülsen (1) zusammen mit jeweils zwei Spezialpapieren (2) wie abgebildet in die Rohre der Sitzlehne. Drehen Sie den Schlitz nach hinten.
- 3. Schieben Sie die Vorbauten (3) in die Adapterhülsen.
- 4. Justieren Sie die die Schiebestange in der gewünschten Höhe. Ziehen Sie die Schrauben der Konusklemmung mit 26 Nm fest.
- 5. Drehen Sie Querstange, so dass sich die Ergogriffe in der gewünschten Position befinden. Ziehen Sie die M6 Schrauben über Kreuz mit 9 Nm fest.




Achtung!

Bitte achten Sie darauf den Vorbau nur im Bereich der Adapterhülsen zu klemmen.

Nummer	Beschreibung	ArtNr.	Menge
1	Vorbau, Edelstahl	22957	2
2	Querstange	20203	1
3	Adapterhülse 22,5/25,6 mm	23049	4
4	Ergogriff, Paar	25063	1
5	Spezialpapier Synflex 0,45mm, I=50mm	20146	4

HASE BIKES Hiberniastraße 2 4573 I Waltrop Germany

Phone +49 (0) 23 09 / 93 77-0 Fax +49 (0) 23 09 / 93 77-20 l info@hasebikes.com www.hasebikes.com

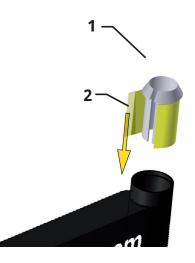



The Push Bar was specially developed for use with Hasebikes trikes (except TRETS). We assume no liability for use with other vehicles.

Installation

- 1. Loosen the upper velcro of the seat cover. Remove the end plugs from the tube ends.
- 2. Push the adapter sleeves (1) combined with each time two tape pieces (2) into the end of tubes in the back of the seat. Rotate the adapter till the gap faces backwards.
- 3. Slide the stems (3) into the adapter sleeves.
- 4. Adjust the sliding bar to the desired height. Tighten the bolts of the conical clamp with 26 Nm.
- 5. Rotate the cross bar so that the ergo grips are in the desired position. Tighten the M6 bolts alternately up to 9 Nm.



Attention!

Please make sure to clamp the stem only in the area of the adapter sleeves.

Number	Description	Art.No.	Quantity
1	Stem, stainless steel	22957	2
2	Crosstube	20203	1
3	Adapter sleeve 22,5/25,6 mm	23049	4
4	Ergo grip, pair	25063	1
5	Tape Synflex 0,45mm, I=50mm	20146	4

www.hasebikes.com #25524 - 03/25