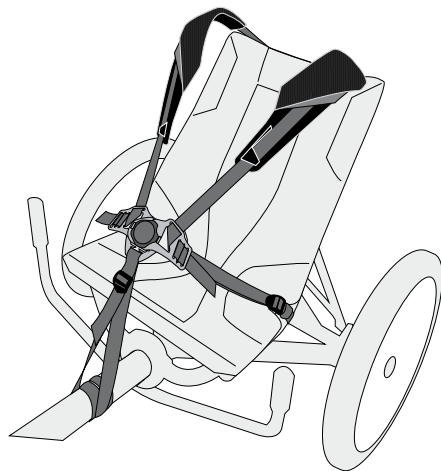
GURTSYSTEM/SCHULTERPOLSTER
HARNESS SYSTEM/SHOULDER CUSHIONING
KOMPONENTEN GURTSYSTEM
COMPONENTS OF THE HARNESS SYSTEM

Gebrauchsanleitung
User's Manual

REF
25535 | 25532

CE **DE**
EN

#25535
#25532

Gurtsystem 06/16 DE-EN

Verwendung

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Intended use

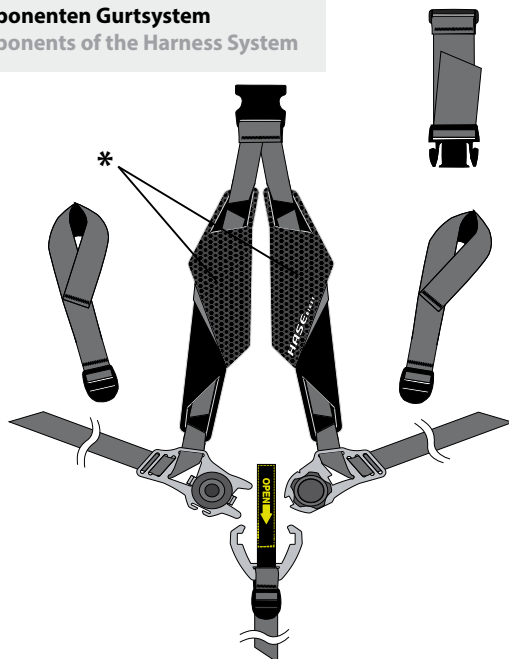
For any rider/passenger at risk of slipping or falling out of the seat when the vehicle is in motion or when the brakes are applied.

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The Harness System was specially developed for use with Hase Bikes. We assume no liability for use of the Harness System with other vehicles. Do not make any modifications to the Harness System. The Harness System may not be used in any manner that adversely affects the safety of the vehicle. When riding, the buckles of the Harness System must be securely fastened at all times. Loose ends of straps must be secured or cut off. Otherwise, they could get caught in moving components. All installation work and adjustments must be performed by a qualified professional. Consult your authorized dealer.

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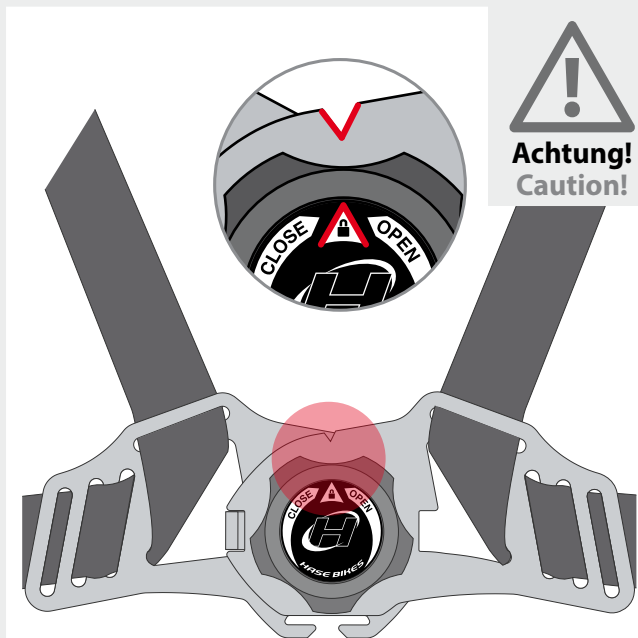
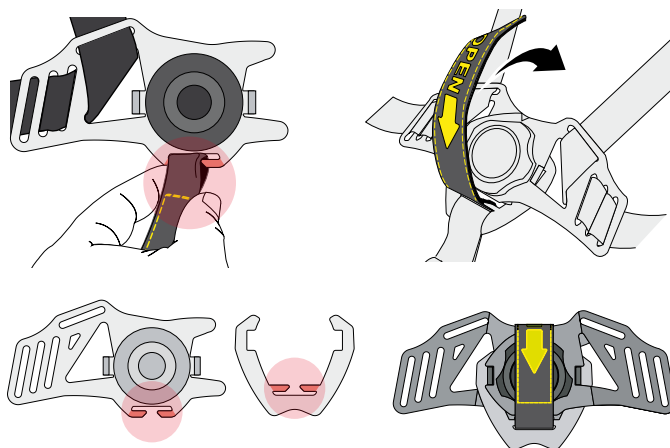
The buckle can be protected with a special Velcro strap, to protect the opening from unexpected use.

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


The buckle is securely fastened when the arrow is pointing upwards.

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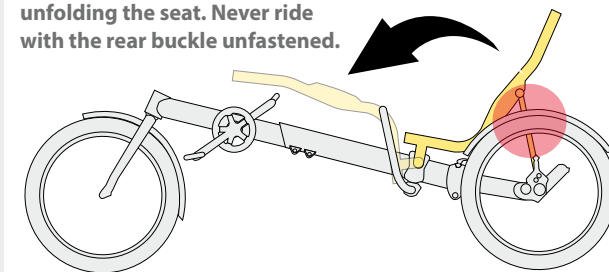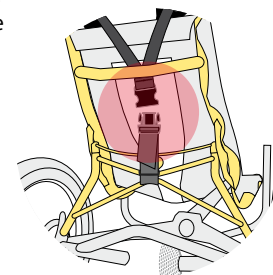
On trikes, the buckle on the rear strap of the harness (behind the seat) must be opened before the seat can be folded. **Caution: Be sure to refasten the buckle when unfolding the seat. Never ride with the rear buckle unfastened.**

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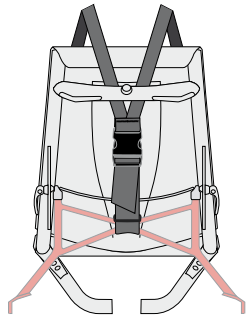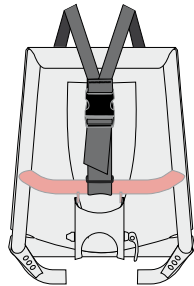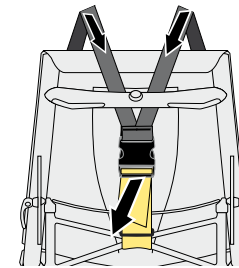
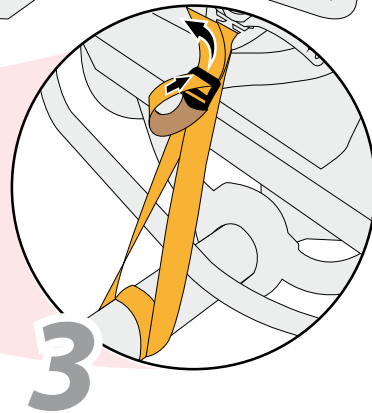
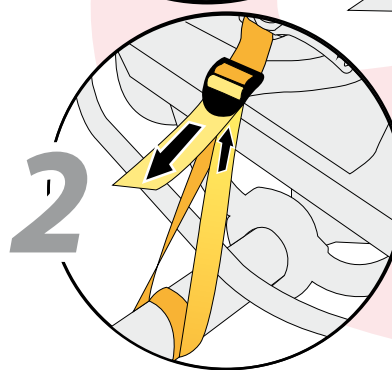
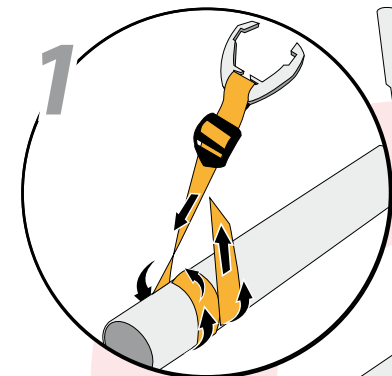
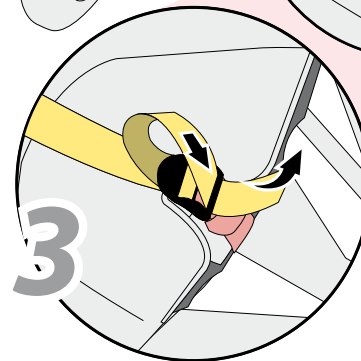
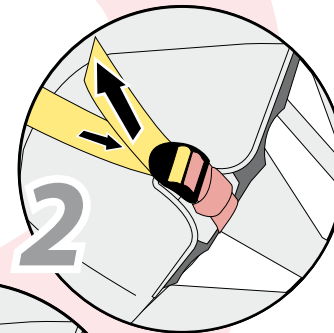
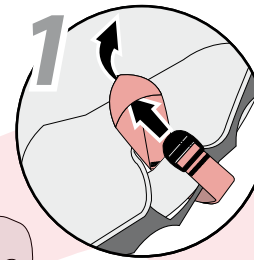
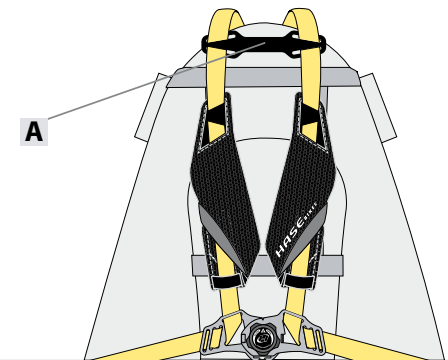
HASE BIKES
Hiberniastraße 2, 45731 Waltrop, Germany

info@hasebikes.com
www.hasebikes.com

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Harness System/Shoulder Pads and Harness System Components

The intended use of the Harness System is for holding the patient in the proper position in the seat. The Harness System corrects poor posture and prevents the patient from slipping out of the seat.

Intended use

The Harness System is intended for riders who require support in order to remain in an upright position in the seat. The reason can be an impaired ability to sit unsupported.

Contraindications

The 5-point Harness is not suitable for any purposes that are contrary to its intended use or that could be hazardous, for example, use as a seatbelt in an automobile or use by persons not familiar with its operation. The Harness must be adjusted to the user. Please observe the contraindications specified in the user manuals of the recumbent bikes and trikes.

Safety Guidelines

The 5-point Harness is only intended for use as a means of ensuring the proper sitting position in the seat of the trike and may not be used as a seatbelt in an automobile.

Defective or damaged Harnesses may not be used and must be immediately replaced.

Reuse

Under normal conditions of use, this accessory can be reused by a new owner after inspection by an authorized dealer.

Protection against environmental influences

The 5-point Harness is made with corrosion-resistant materials. Anodized aluminum parts, plastics, and stainless steel are used.

Cleaning and disinfection

The 5-point Harness can be cleaned and disinfected using common household cleaning and disinfecting agents.

Applicable standards and guidelines

This is an accessory within the meaning of Regulation 2017/745 on medical devices, which as such is CE-marked.

Information regarding incidents

Any serious incident* that occurs in connection with the product must be reported to the manufacturer and the responsible competent authorities of the member state in which the user and/or patient resides.

* A "serious incident" is any incident that had, could have had, or could have one of the following consequences, either directly or indirectly: a) the death of a patient, user, or other person, b) the serious temporary or permanent deterioration of the health condition of a patient, user, or other person, c) a serious risk to public health.

Whom to contact with questions or problems

If you have any problems or questions about your therapeutic bike or trike, please contact your authorized dealer or HASE BIKES for assistance.

Residual risks

None known

Service life

Depending on the frequency of use, a service life of up to 8 years can be expected if the product is used in accordance with the user manual.

Disposal instructions

At the end of its useful life, the product should be disposed of properly, in accordance with the local waste disposal regulations (i.e. not as household waste).