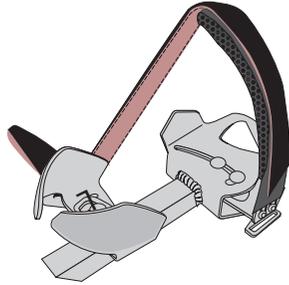
## SPEZIALPEDALE SPECIAL PEDALS

Gebrauchsanleitung  
User's Manual

REF  
27160.RI | 27160.LE  
25610.RI | 25610.LE  
25583.RI | 25583.LE

#25583.LE  
#25583.RI

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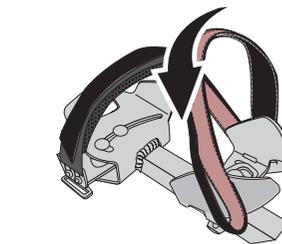
If the foot is positioned far forward on the pedal, loosen the bolts **B** and reposition the strap holder accordingly.

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
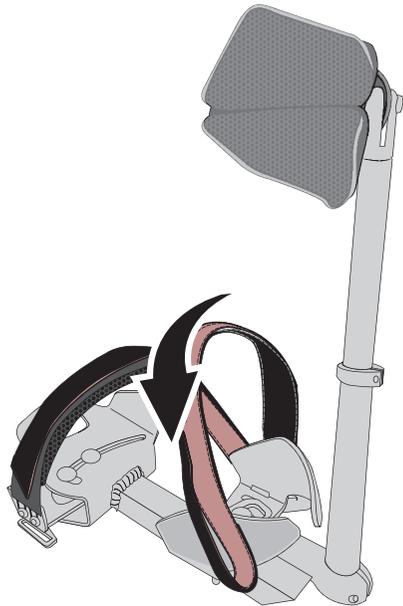
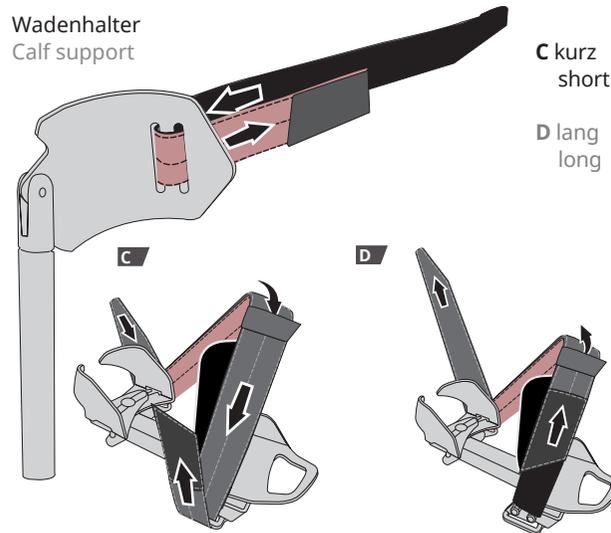
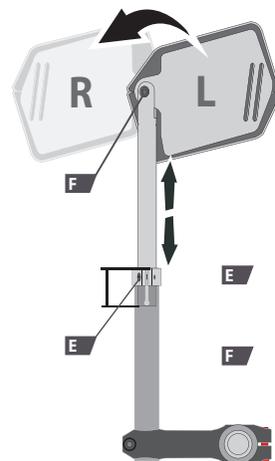
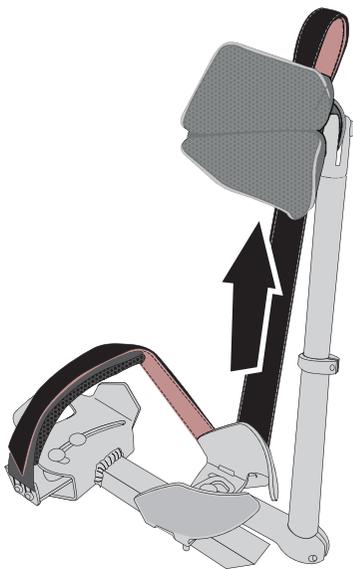
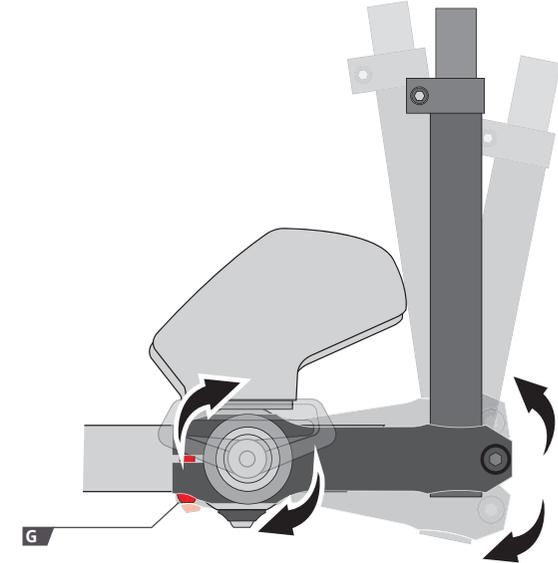
From a medical perspective, movement is important for healthy joints. This is why the mounting mechanism for the calf support uses a flexible elastomer spring that allows movement but returns the support to the middle position for easier handling. If desired, the calf support can also be set to a fixed position by tightening the bolt **G**.

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
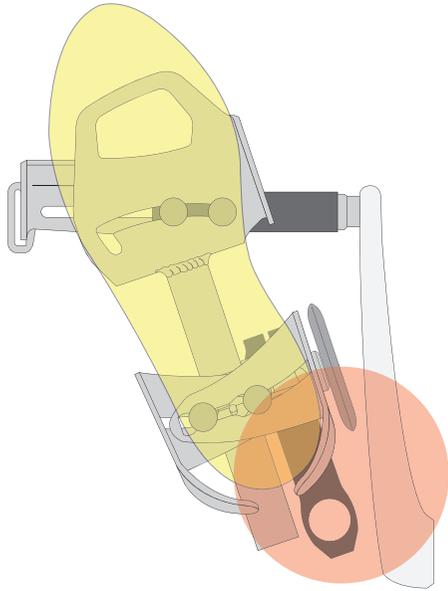
When making adjustments, be sure to check that no parts of the pedals or calf supports can hit or rub against the crank arms when the rider is pedaling.

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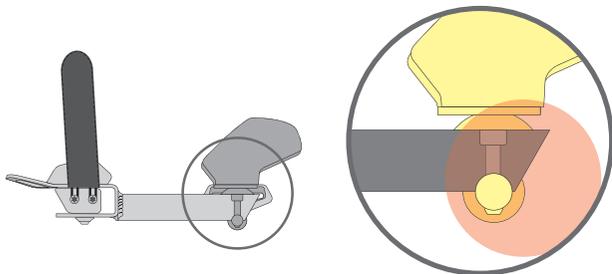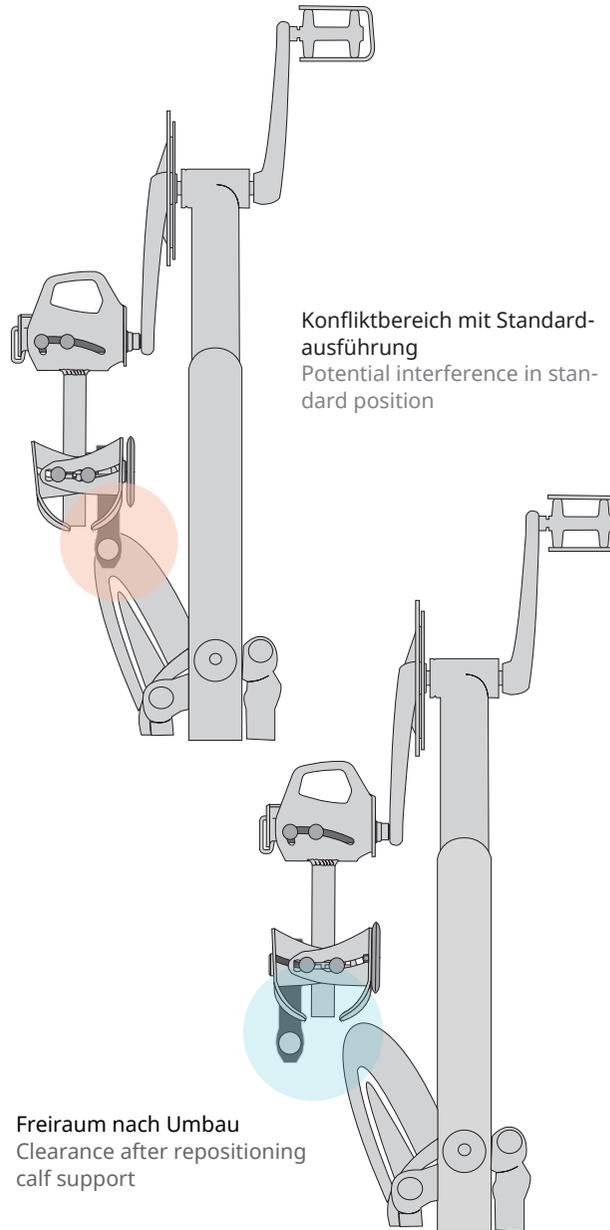
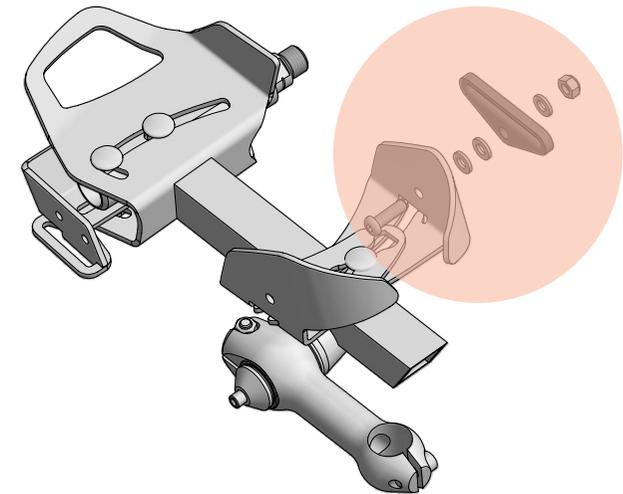
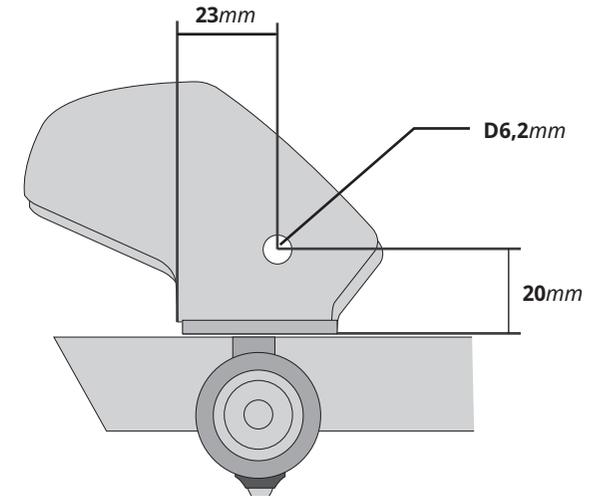
Assembly with calf support in standard position

## Umbau

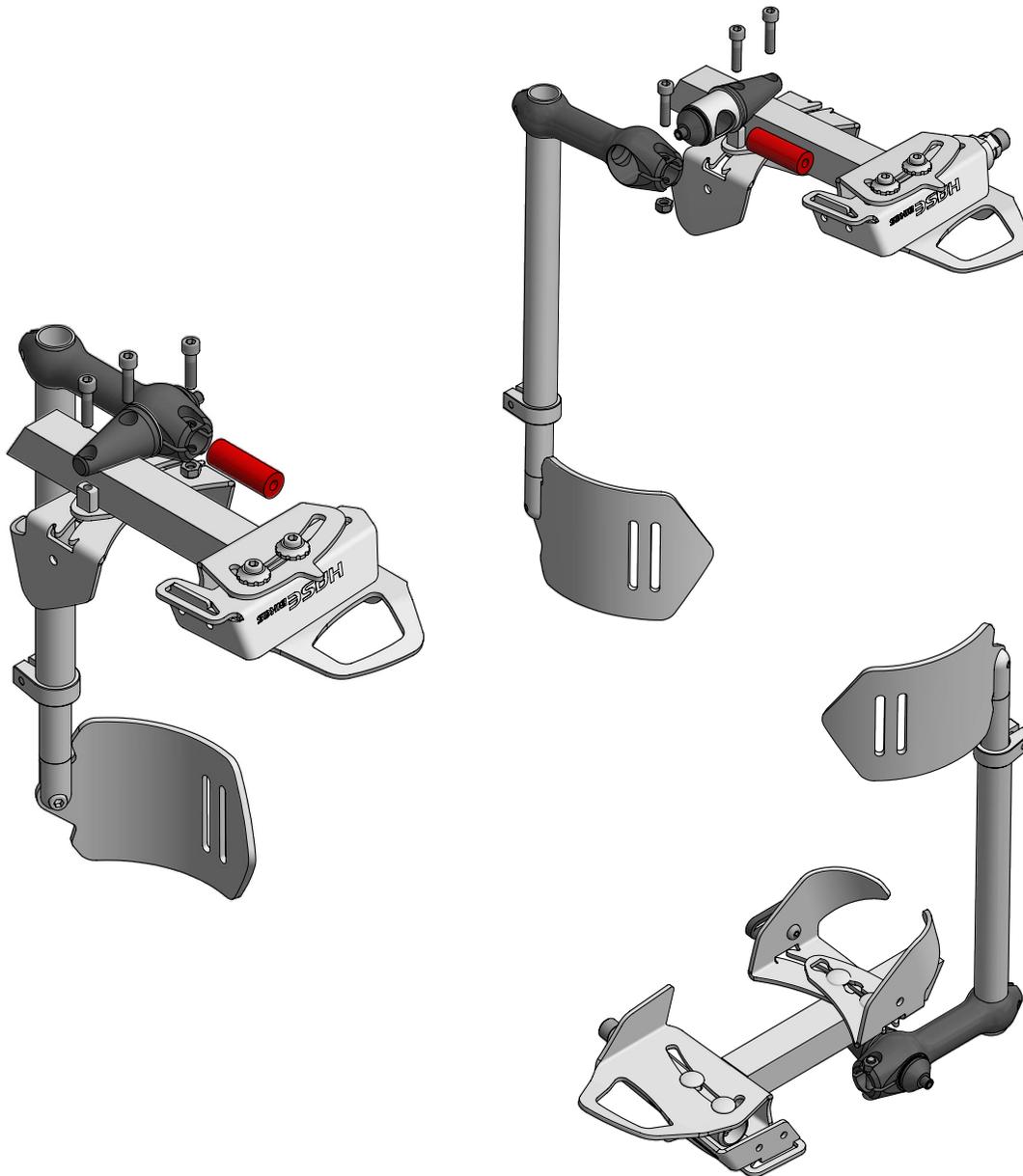Assembly with calf support in alternative position

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Prior to riding with these pedals for the first time, be sure to ask your doctor or physical therapist whether the use of the pedals or cycling in general poses any risks to your health. All installation work and adjustments must be performed by a qualified professional. Consult your authorized dealer. Do not make any modifications to the pedals. The pedals may not be used in any manner that adversely affects the safety of the vehicle. The pedals can only be used for multi-track vehicles (e.g. trikes) or for the front rider of the semi-recumbent tandem Pino.

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### Special Pedals

These pedals are intended for use by riders with foot deformities and/or motor disorders.

### Intended use

The Special Pedals are intended for use by riders with foot deformities. The Pedal with Calf Support is suitable for persons with severe internal or external rotation of the leg.

### Contraindications

The Special Pedal (with or without Calf Support) is not suitable for any purposes that are contrary to its intended use or that could be hazardous. Please observe the contraindications specified in the user manuals of the recumbent bikes and trikes.

### Safety Guidelines

Defective or damaged pedals may not be used and must be immediately replaced.

### Reuse

Under normal conditions of use, this accessory can be reused by a new owner after inspection by an authorized dealer.

### Protection against environmental influences

The Special Pedals are made with corrosion-resistant materials. Powder-coated and anodized aluminum parts, plastics, and stainless steel are used.

### Cleaning and disinfection

The Special Pedals can be cleaned and disinfected using common household cleaning and disinfecting agents.

### Applicable standards and guidelines

This is an accessory within the meaning of Regulation 2017/745 on medical devices, which as such is CE-marked.

### Information regarding incidents

Any serious incident\* that occurs in connection with the product must be reported to the manufacturer and the responsible competent authorities of the member state in which the user and/or patient resides.

\* A "serious incident" is any incident that had, could have had, or could have one of the following consequences, either directly or indirectly: a) the death of a patient, user, or other person, b) the serious temporary or permanent deterioration of the health condition of a patient, user, or other person, c) a serious risk to public health.

### Whom to contact with questions or problems

If you have any problems or questions about your therapeutic bike or trike, please contact your authorized dealer or HASE BIKES for assistance.

### Residual risks

None known

### Service life

Depending on the frequency of use, a service life of up to 8 years can be expected if the product is used in accordance with the user manual.

### Disposal instructions

At the end of its useful life, the product should be disposed of properly, in accordance with the local waste disposal regulations (i.e. not as household waste).